FKG

swiss endo

Rooter

OPERATION INSTRUCTIONS

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Using your Rooter motor

- 1 - Press one of the four buttons to switch on your device. Make sure that the displayed torque (TQ) and speed (Rpm) correspond to the parameters of the used files. Change, if necessary.
- 2 - Press the **ON/OFF** button to start your motor rotating. Press the **ON/OFF** button again to stop your motor (Pulse function).
- 3 - Pressing and holding down the **ON/OFF** button for more than one second will start your motor rotating ; releasing the button will stop the motor. (Rotation while pressed).

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Switching off your Rooter motor

! Press the " S " button until the indicator lights go off.

! If none of the buttons are pressed during three minutes, your device shuts off.

! If your motor runs without any charge for three minutes, your device shuts off.

! If your device is off and you press and hold down the ON/OFF button, your device will switch on and your motor will start rotating.

! If your motor is charged while rotating, it will not shut off.

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Installing and removing the LED holder.

1 Install the LED holder under the contra angle head.

2 Raise the LED holder to place it flush under the contra angle head.

3 Reverse the steps of the installing operation to remove the LED holder.

! The white LED can be removed before or simultaneously with the contra angle head.

! The contra angle head can be installed in 5 other positions at 30° intervals. In this case, the white LED cannot be installed.

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Adjusting the micromotor torque

- 1- Press the " S " button **once** (The Torque indicator light flashes - Fig. 1) ; the other indicator lights go off (Except for the battery and the display),
 - 2- Press the " + or - " button, to increase or decrease the torque. The programming stops after three seconds (the torque indicator light is now permanently lit and the other indicators go on).
- ! Warning : this operation changes the torque value of the selected memory.

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Adjusting the micromotor speed

1- Press the " S "button **twice** (The Speed indicator light flashes - Fig. 1) ; the other indicator lights go off (Except for the battery and the display),

2- Press the " + or - " button to increase or decrease the speed. The programming stops after three seconds (the speed indicator light is now permanently lit and the other indicator lights go on).

! Warning : this operation changes the speed value of the selected memory.

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Adjusting the sound signal

1 - Press the "S" button **three times**.

2 - The beep's indicator light flashes ; the other indicator lights go off (except for the battery).

3 - The display indicates the Beep's force : "0" No Beep (Fig. 1), "1" Average Beep (Fig. 2), "2" Max Beep (Fig. 3).

4 - Press the " + or - " button to increase or decrease the Beep (a sound is emitted during the adjustment except if "0" is displayed). The programming stops after three seconds (the Beep's indicator light is now permanently lit and the indicator lights of the other parameters such as torque, speed, etc. go on).

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Adjusting the LED switch-off time (in seconds)

1- Press the "S" button **four times**.

2 - The display flashes; the other indicator lights go off (except for the battery). The display indicates the LED switch-off time after the motor stops : example : "5" = 5 sec. (Fig. 1), "0" = 0 sec. (Fig. 2), "9" = 9 sec. max (Fig. 3).

3 - Press the "+" or "-" button, to increase or decrease the switch-off time. The programming stops after three seconds (the display is now permanently lit and the indicator lights of the other parameters such as torque, speed, etc. go on).

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Recharging the batteries

1 – Place your motor on its charger. (Fig.1)

2 – During the charging operation, two indicator lights go on (one green and one orange). (Fig.1)

3 – At the end of charging, the orange indicator light goes off and the green indicator light indicates that the charging operation is terminated (Fig.2).

In case of a battery failure, a red indicator light will go on instead of the green indicator light. Lift up your motor to cut off the charging and then reinstall it ; a new charging cycle starts. If the red indicator light goes on again, replace the battery.

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! ▲Recharging the batteries

- Carefully place your motor on its charger to not damage the contacts.
- Use only the charger supplied with your motor.
- The batteries temperature is checked and can cut off the charging if the temperature rises above 40°C ; charging will resume when the temperature drops below 40°C.
- Do not leave the charger or the motor in front of a window exposed to bright sunlight.
- The motor cannot be used if it is installed on its charger.

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Selecting and deselecting the automatic torque return

- 1 - By default the automatic torque return is activated. (Fig. 1)
 - 2 - Press the " - " button approximately 1 second to deactivate the automatic return. (Fig. 2)
 - 3 - The indicator light goes off.
 - 4 - Press the " + " button approximately 1 second to activate the automatic return.
 - 5 - The indicator light goes on.
- ! When the motor starts rotating, pressing the " + or - " button, immediately activates or deactivates the automatic return.

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Information on the max speed / max torque ratio

The max speed and the max torque are interconnected (see the table : The authorised work area in the blank sections).

- The choice of a max speed > 400 rpm can automatically reduce your device's torque.
- Conversely, the choice of a max torque > 0.75 Ncm can automatically reduce your device's speed.

* Warning : To obtain the maximum speed of 600 to 1200 rpm, your battery should be well charged.

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Lubricating the contra angle before each sterilisation.

- Screw the lubricating adapter (1) by turning it approximately 10 times on the tip of your spray bomb * (Fig. 1)
 - Insert the contra angle head (2) on the lubricating adapter (Fig. 2). During lubrication, hold the contra angle head to prevent it from being ejected and therefore damaged.
 - Vaporise for 2 to 3 seconds maximum. The oil traverses, cleans and lubricates the entire contra angle head. (Fig. 2)
 - Remove excess lubricant before mounting the contra angle head on the motor.
- * hygienic pharmaceutical sterilisation lubricant for the dental handpiece.

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Cleaning and sterilisation.

- Clean the motor and charger only with the colourless alcohol-based disinfectant sold in shops (ethanol, isopropanol). Use of a curing powder or an abrasive sponge would damage its surface.
 - The motor and charger are not protected against water splashes. They cannot be sterilised. Do not spray a cleaning product on the motor or the charger. - Do not lubricate the motor. - Do not use disinfectant wipes.
 - Only the contra angle head and the LED can be sterilised. Sterilise before using them for the first time and after each patient.
- For the contra-angle, refer to the NSK sterilisation protocol. Reminder: lubricate the contra-head. Remove the excess lubricant: In a bag and in an autoclave at 134°C for 18 minutes (Prion cycle).

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Cleaning, disinfecting and sterilisation

Equipment: brush, cloth or disposable wipe.

Method: remove excess dirt on the LED holder. In a bag and in an autoclave at 134°C for 18 minutes (Prion cycle). Put a maximum of 3 LEDs in a bag suitable for autoclave use.

Drying: no drying on removal from the autoclave.

Maintenance: no particular maintenance for this appliance. No lubrication.

Inspection and operating tests: check that the LED lights when the motor is rotating.

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Warning

The LED holder should not be immersed in cold disinfectant solution, nor in an ultrasonic tank, could this damage the LED holder contacts. Do not exceed 134°C. Do not use detergents.

When sterilising more than one instrument in a single autoclave cycle, make sure that the maximum load of the autoclave is not exceeded.

Retreatment limits

Repeated sterilisation processes have little effect on these instruments. The end of life is normally determined by wear (40 to 60 sterilisation cycles) and damage due to use.

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Accident prevention

Many maintenance and operating problems with the Router are caused by a lack of attention to elementary safety precautions. Problems and accidents can be avoided by following the manufacturer's recommendations Start by reading all cautions and instructions concerning safety and accident prevention thoroughly. Then run the equipment taking great care to prevent other damage to the equipment itself and to prevent injury

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FKG is not responsible for accidents, material damage or injury resulting from:

- a repair by a person not authorised by FKG.
- a change, modification or deterioration of the product.
- the use of products or equipment produced by other manufacturers, except for those supplied by FKG.
- maintenance or repair using spare parts or components other than those recommended by FKG.
- operations other than those defined in the descriptive procedure of this manual or resulting from the non-observation of the precautions and accident prevention described in this manual.
- working, storage or installation conditions such as, for instance, a supply current not in conformity with the specifications.

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Users, that is dentists, are responsible for the maintenance and correct operation of this medical appliance. These medical appliances should only be used by members of the dental profession. FKG does not approve any repairs carried out by a person not certified by FKG.

This appliance must not be connected to, or used in combination with, another appliance or system. FKG is not responsible for accidents, material damage, injury or other problems resulting from ignorance of these clauses.

Never allow the appliance to fall or expose it to any type of impact or shock.

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Precision in the measurement of the widening of the canal is not always possible depending on the shape and the condition of the tooth, as described in the user guide. Stop using the appliance immediately if you feel a tactile reaction suggesting abnormal conditions.
Make sure that the file is fully inserted in its housing. Pull on the file slightly to make sure that it is properly fitted.
Never press the contra angle push button while the motor is running.
Take care not to injure your finger when inserting or removing the file from its housing.
Stainless steel files cannot be used with this appliance.

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Titanium nickel files require care to be taken about the points listed below to avoid breakages.

Never use force to insert the file into the contra angle.

Never use force to make the file enter the canal.

Follow the file manufacturer's directions: clinical protocol, duration of use, etc.

Never use damaged files.

To work efficiently, you need to first acquire practice and experience. We recommend that the dentist practices on extracted teeth or root models.

DO NOT USE THE ROOTER FOR OTHER SITUATIONS THAT ARE NOT SPECIFIED.

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Technical Characteristics	
Manufacturer	FKG Dentaire SA, CH-2304 La Chaux-de-Fonds- Suisse
Model	Rooter
Classification EN 60601-1	Class B, Applied part of type BF, IP 30
Classification 93/42 EEC	Class IIa
Power supply	1 rechargeable battery Li-Ion 3.7 Volts 800 mAh 300 to 500 charge and discharge cycles. Your motor is delivered with an uncharged battery. Charge before using it for the first time.. For used batteries: Please dispose of these through the waste recycling scheme applicable in your country.
Charger	Input : 100-240VAC / Output : 9 Volts - 0.6 A Cord : 1.5 metres
1:1 contra angle	Maximum rotation speed : 1200 rpm Tools, diameter 2.35 - type 1 Special coupling for Rooter Autoclavable at 134°C
White LED holder	Autoclavable at 134°C - 40 to 60 cycles in an autoclave.
Motor box and charger holder	Made of white ABS
Weight / Dimensions in mm (HxLxw)	134 g / 27 x 199 x 23
Operating conditions	Temperature range : +10°C to + 35°C Relative humidity : 30% to 75%
Storage and transport conditions	Temperature range : -10°C to + 40°C Relative humidity : 30% to 95%

After Sales Service

For all maintenance or repairs, send your appliance to your distributor or directly to the FKG Service department (our address is on www.fkg.ch). For a return under warranty please attach a copy of your invoice; this will help your request to be handled more quickly.

Before dispatching your equipment, disinfect and sterilise it as explained in the user instructions.

Wrap your appliance carefully before dispatching it; we cannot accept responsibility in the event of damage during transport.

- To completely overhaul your device, please join all the parts of your motor.
- Customer returns will only be accepted for instruments that are cleaned, disinfected, sterilised (for the contra-angle, the LED and the files) and satisfactorily packed. If not, the instruments will unfortunately have to be returned to the sender as received.

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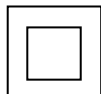
Electromagnetic emissions

1.6 Test Conditions and Results - Conducted Emissions

CISPR 11	TEST: Limits of mains terminal disturbance voltage	Verdict
<p>Method: The AMN placed 0,8 m from the boundary of the unit under test and bonded to a ground reference plane. This distance was between the closest points of the AMN and the EUT. All other units of the EUT and associated equipment were at least 0,8 m from the AMN. All power was connected to the system through Artificial Mains Network (AMN). Conducted voltage measurements on mains lines were made at the output of the AMN.</p>		P

Laboratory Parameters	Required prior to the test	During the test
Ambient Temperature	10 to 40 °C	22 °C
Relative Humidity	10 to 90 %	57%
Fully configured sample scanned over the following frequency range	Frequency range on each side of line	Measurement Point
	150kHz to 30MHz	Mains

Limits - Group 1 and 2 - Class B - EUT Configuration mode 2

Frequency (MHZ)	Limit dB (uV)			
	Quasi Peak	Result *	Average	Result *
0,15 to 0,50	66 to 56	12	56 to 46	2
0,50 to 5	56	7,1	46	0,2
5 to 30	60	16,8	50	6,8

Supplementary information: The result in this table is the minimum margin to the limit.

Electromagnetic emissions

1.7 Test Conditions and Results - Radiated Emissions		
CISPR 11	TEST: Limits for radiated disturbance 30 MHz - 1 GHz	Verdict
<p>Method: Measurements were made in a 10-meter Open Area Test Site that complies to CISPR 16. Preliminary (peak) measurements were performed at an antenna to EUT separation distance of 10 meter. The EUT was rotated 360° about its azimuth with the receive antenna located at various heights in horizontal and vertical polarities. Final measurements (quasi-peak) were then performed by rotating the EUT 360 and adjusting the receive antenna height from 1 to 4 m All frequencies were investigated in both horizontal and vertical antenna polarity where applicable.</p>		P
Laboratory Parameters	Required prior to the test	During the test
Ambient Temperature	10 to 40 °C	20 °C
Relative Humidity	10 to 90 %	57%
Fully configured sample scanned over the following frequency range	Frequency range on each side of line	Measurement Point
	30MHz to 1GHz	10 m measurement distance
Limits - Group 1 - Class B - EUT Configuration mode 1		
Frequency (MHZ)	Limit dB (uV/m)	
	Quasi Peak	Results *
30 to 230	30	4dB
230 to 1000	37	No frequency observed
Limits - Group 1 - Class B - EUT Configuration mode 2		
Frequency (MHZ)	Limit dB (uV/m)	
	Quasi Peak	Results *
30 to 230	30	3dB
230 to 1000	37	No frequency observed
Supplementary information: The result in this table is the minimum margin to the limit		

Electromagnetic emissions

1.8 Test Conditions And Results - Limits for Harmonic Current Emissions		
61000-3-2	TEST: Limits for Harmonic current emissions (IEC 61000-3-2: - use latest edition)	Verdict
<p>Method: This test consists on the measurement of harmonics components of the input current which may be produced by equipment having an input current up to and including 16 A per phase, and intended to be connected to public low-voltage distribution systems. The equipment is tested under specified conditions of operation.</p>		P
Laboratory Parameters	Required prior to the test	During the test
Ambient Temperature	15 to 35 °C	-
Relative Humidity	30 to 60 %	-
Equipment mode	Power interface mode	230V 50Hz
	EUT configurations mode	2
	Operation mode	2
Classification of Equipment		Class A
<p>Supplementary information: This 10W equipement is compliant for harmonic current emissions without measurement. (p<75W)</p>		

Electromagnetic emissions

1.9 Test Conditions and Results - Limitation of Voltage Fluctuations and Flicker		
61000-3-3	TEST: Limitation of Voltage Fluctuations And Flicker (IEC 61000-3-3: - use latest edition)	Verdict
<p>Method: The test circuit consists of a test supply voltage, reference impedance, the equipment under test and a flicker meter compliant with IEC 60868 The equipment shall be tested in the condition in which the manufacturer supplies it.</p>		P
Laboratory Parameters	Required prior to the test	During the test
Ambient Temperature	15 to 35 °C	-
Relative Humidity	30 to 60 %	-
Equipment mode	Power interface mode	230V 50Hz
	EUT configurations mode	2
	Operation mode	2
<p>Supplementary information: This equipment is unlikely to produce voltage fluctuation and flicker. This equipment is compliant without</p>		

Electromagnetic emissions

1.10 Test Conditions and Results - Immunity to Electrostatic Discharges

61 000-4-2	TEST: Electrostatic discharges (IEC 61000-4-2:- use latest edition)		Verdict
<p>Method: The test is intended to demonstrate the immunity of equipment subjected to static electricity discharges from operators directly and to adjacent objects. The tabletop equipment under test is placed on a 0.8 m on the table. The EUT and the cables are isolated from the coupling plane by an insulating support 0.5 mm thick. The floor standing equipment is isolated from the ground reference plane by an insulating support about 0.1 m thick. The vertical coupling plane (VCP) of dimensions 0.5 m x 0.5 m is placed parallel to, and positioned at a distance of 0.1 m from, the EUT.</p>			P
Laboratory Parameters	Required prior to the test		During the test
Ambient Temperature	15 to 35 °C		21 °C
Relative Humidity	30 to 60 %		50%
Test Levels			
Discharge type	Discharge Level (kV)		Number of discharges per location (each polarity)
	Positive	Negative	
Air-Direct	2,4,8	2,4,8	10
Contact-Direct	2,4,6	2,4,6	10
Contact-Indirect	2,4,6	2,4,6	10
Discharge type	See photo documentation of the test set-up All external locations accessible by hand Horizontal plate (HCP) Vertical coupling plate (VCP)		
Supplementary information: EUT powered at one of the Nominal input voltages and frequencies: 3.6V DC in configuration mode 1			

Electromagnetic emissions

1.11 Test Conditions and Results - Immunity to Radio Frequency Electromagnetic Fields			
61 000-4-3	TEST: RF electromagnetic fields (IEC 61000-4-3: - use latest edition)		Verdict
Method: The test allows estimating of the radiated immunity of electrical and electronic equipment to electromagnetic disturbances coming from intended radio-frequency (HF) transmitters in the frequency range 30 MHz to 1000 MHz. The interference is applied on the enclosure of the equipment by using transmitting antennas.			P
Laboratory Parameters		Required prior to the test	During the test
Ambient Temperature		15 to 35 °C	21 °C
Relative Humidity		30 to 60 %	49%
Test specifications			
Calibration Requirements	Uniform field area (UFA)	1.5 m x 1.5 m, 16 points with a minimum UFA size 0.5 m x 0.5 m	
		75 % of calibration points within specifications if UFA is larger than 0.5 m x 0.5 m. 100 % (all 4 points) in the specifications for 0.5 x 0.5 m UFA	
Frequency bandwidth		80 MHz to 2500 MHz	
Level	Non-Life Supporting Equipment	3V/m	
		Amplitude modulation	80 % / 1 KHz sine wave
			900 MHz and 1800 MHz Pulse at 80% AM - 200Hz
			Controls, monitors or measures a physiological parameter, (80 % / 2 Hz)
Level	Life Supporting Equipment	10V/m	
		Amplitude modulation	80 % / 1 KHz sine wave
			900 MHz and 1800 MHz Pulse at 80% AM - 200Hz
			Controls, monitors or measures a physiological parameter, (80 % / 2 Hz)
Frequency step		1% with 3s dwell time	
Supplementary information: EUT powered at one of the Nominal input voltages and frequencies: 3.6V DC in configuration mode 1.			

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Warranty

The warranty is valid from the date of purchase of your product for a period of one year. Failure to follow the instructions of the manual and wear parts are excluded from the warranty. Any work on your appliance by a person not approved by FKG voids the warranty. Damage due to not following instructions or normal wear of parts are excluded from the warranty. When returning the appliance under warranty, attach a copy of your invoice for faster processing of your request. Before sending your unit back, clean, disinfect and sterilize it according to instructions. So that the job can be dealt with quickly, please return the warranty card enclosed with these instructions that you received with your machine.

Carefully pack your appliance before sending it, because we cannot be held liable for any damage during transport.

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